

PCKE SALIVA EJECTOR PRODUCT INSTRUCTIONS

CAUTION: Saliva Ejector is to be used with PCKE Cheek Expander Only

The PCKE Saliva ejector System provides cheek expansion and saliva evacuation.

Included in each Kit is the following:

- One Saliva ejector tube with quick disconnect Leur
- One Saliva Ejector
- One PCKE

STEP 1:

Attach saliva ejector to PCKE as shown in figure:1.



Figure 1
PCKE and Saliva Ejector in Position for Assembly

STEP 2:

Fold saliva ejector over PCKE



Figure 2 Assembly Step 2

STEP 3:

Fold over saliva ejector until it snaps into place as shown below in figure 3.



Figure 3 Saliva Ejector Fastened to PCKE

STEP 4:

Inset tube onto barbed tubing fitting on saliva ejector.



Figure 4 Tubing applied to Saliva Ejector Barb. The PCKE Saliva ejector is now fully assembled and ready to be used.